


Assembly Aid for Cover Plate for NEURO SPRING and NEURO VARIO-SPRING System Ankle Joints

Assembling a Joint with the Assembly Aid for the Cover Plate

The assembly aid for the cover plate facilitates assembling ankle joints with foot lifter. Use the assembly aid for the cover plate for an easier tensing of the pressure spring in the functional unit.

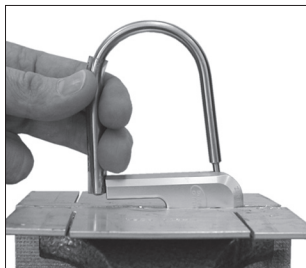

1. Clamp the functional unit by using a vice with braces. Position the narrow end of the assembly aid for the cover plate below the ball of the functional unit and fix the plastic-sheathed side on the upside of the spring duct by tensing the spring.



2. After mounting the system stirrup mount the functional unit that is pre-tensioned with the assembly aid. Put on the cover plate laterally on the joint's lower part and the system stirrup and screw in the countersunk screws.



3. If the system stirrup moves freely and the cover plate is assembled, remove the assembly aid for the cover plate by pulling off the plastic-sheathed side laterally.

FIOR & GENTZ
ORTHOPÄDIETECHNIK MIT SYSTEM