

## “Knock knock” Gesture for the User App on Your Samsung Galaxy Watch

The “Knock knock” gesture allows you to use the User app on your Samsung Galaxy Watch more easily – through a simple hand gesture.

You can perform the following actions using the “Knock knock” gesture:

- start the User App
- temporarily unlock the system knee joint in Auto mode to sit down

You have the option to activate the “Knock knock” gesture on your Samsung Galaxy Watch either via your Android smartphone or Samsung Galaxy Watch.

If you’re using your Android smartphone to activate the gesture, use the instructions under **A**. (Make sure your smartphone and Samsung Galaxy Watch are coupled to be able to use this option.)

If you’re using your Samsung Galaxy Watch to activate the gesture, use the instructions under **B**.


## Set up the “Knock knock” Gesture for the User App on Your Samsung Galaxy Watch

If you haven’t already, connect to your orthosis with the User app on your Samsung Galaxy Watch following the instructions in the User app.

### A. Setup on Your Android Smartphone

Set up the “Knock knock” gesture on your **Android Smartphone** using the **Galaxy Wearable App**

1. Start the **Galaxy Wearable** app  on your phone.
2. Go to the **Watch Settings**  in the app.
3. Go to **Buttons and gestures > Knock knock**, then turn on.
4. Tap the item **Select an option**, then swipe to **User** and tap it.

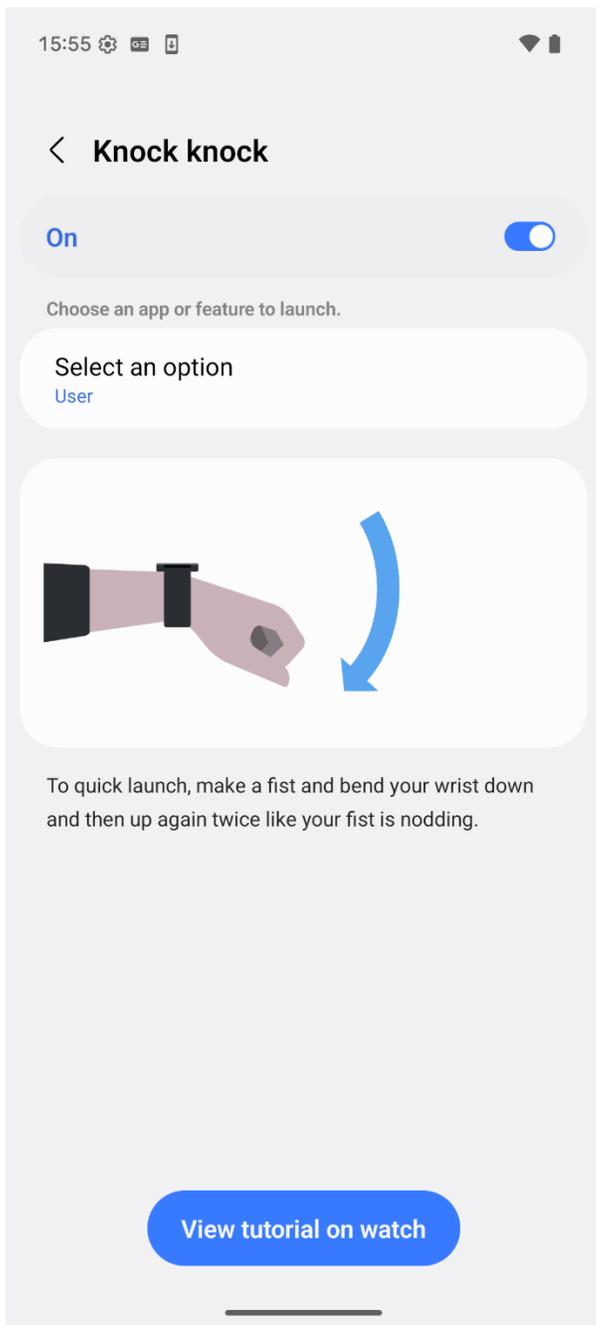

**Tip:** to learn how to use the “Knock knock” gesture, tap **View tutorial on watch** below the **Select an option** item. An interactive animation shows you how to perform and perfect the gesture.

## B. Setup on Your Samsung Galaxy Watch

### Set up the “Knock knock” Gesture on Your **Samsung Galaxy Watch**

1. Go to the **Settings App**  on your Samsung Galaxy Watch.
2. Go to **Buttons and gestures > Knock knock**, then turn on.
3. Tap the item **Select an option**, then swipe to **User** and tap it.



**Tip:** to learn how to use the “Knock knock” gesture, tap **Try it** below the **Select an option** item. An interactive animation shows you how to perform and perfect the gesture.

## Use the “Knock knock” Gesture with the User App on Your Samsung Galaxy Watch

Put the orthosis on and set it into Auto mode. Use the “Knock knock” gesture with the User App by following these steps:

1. Wake the Samsung Galaxy Watch display (Raise your wrist).
2. Use the “Knock knock” gesture to open the User app.
3. The User app opens and indicates the current mode. If Auto mode is selected, use the “Knock knock” gesture to temporarily unlock the system knee joint to sit down.