


patient commission/configuration ID \_\_\_\_\_

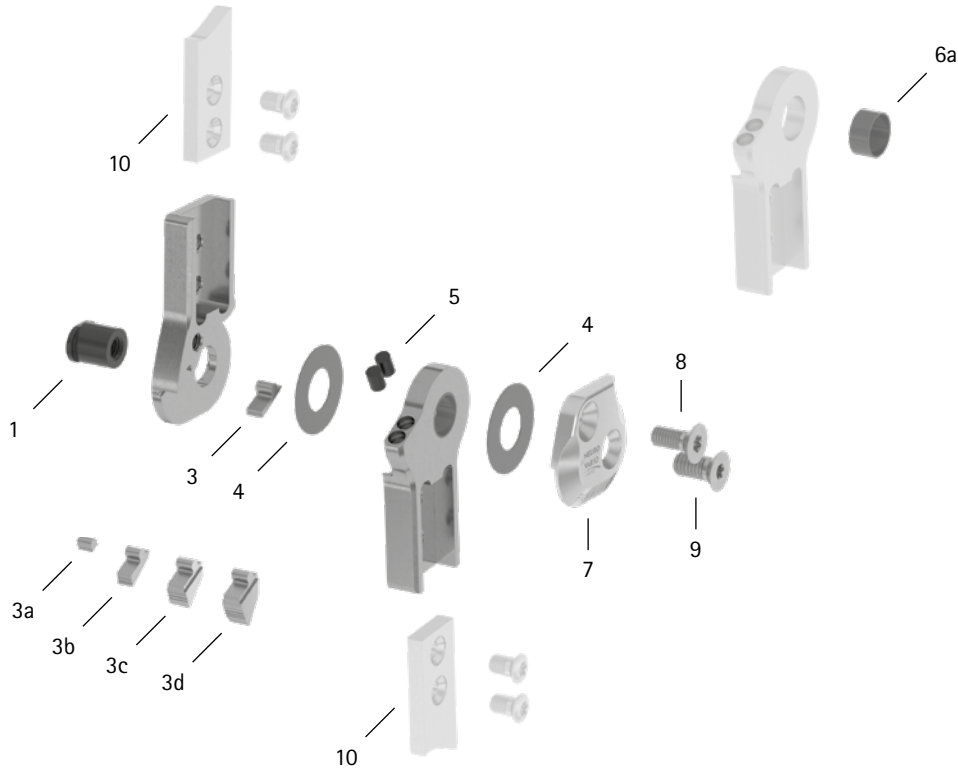

Item	20mm	Description	Potential Problem	Measure	Recommended Inspection, Potential Replacement*	Latest Replacement after
<b>System Joint</b>						
1	SB1069-L1020	bearing nut	wear	replacing	every 6 months	36 months
3a	SJ9025-E000	0° extension stop	wear	replacing	every 6 months	if required
3	SJ9025-E005	5° extension stop				
3b	SJ9025-E010	10° extension stop				
3c	SJ9025-E020	20° extension stop				
3d	SJ9025-E030	30° extension stop				
4	GS2611-040	sliding washer, Ø = 26mm	wear	replacing	every 6 months	18 months
4	GS2611-045					
4	GS2611-050					
4	GS2611-055					
4	GS2611-060					
5	PN1000-L05/5	extension stop damper	wear	replacing	every 6 months	18 months
6a	BP1211-L060	sliding bushing	wear	replacing	every 6 months	18 months
7	SJ0195-L/AL	cover plate, left lateral or right medial, aluminium	wear	replacing	every 6 months	36 months
7	SJ0195-R/AL	cover plate, left medial or right lateral, aluminium				
8	SC1405-L14	countersunk flat head screw with hexalobular socket	wear	replacing	every 6 months	36 months
9	SC1406-L14	countersunk flat head screw, hexalobular socket (axle screw)				
<b>System Anchors and System Side Bars</b>						
10	see catalogue	system anchor and system side bar	wear or breakage	replacing	every 6 months	if required

\* depending on the assessment of the distributor of the custom-made product regarding the patient's usage behaviour

WP0084-GB-2024-04