

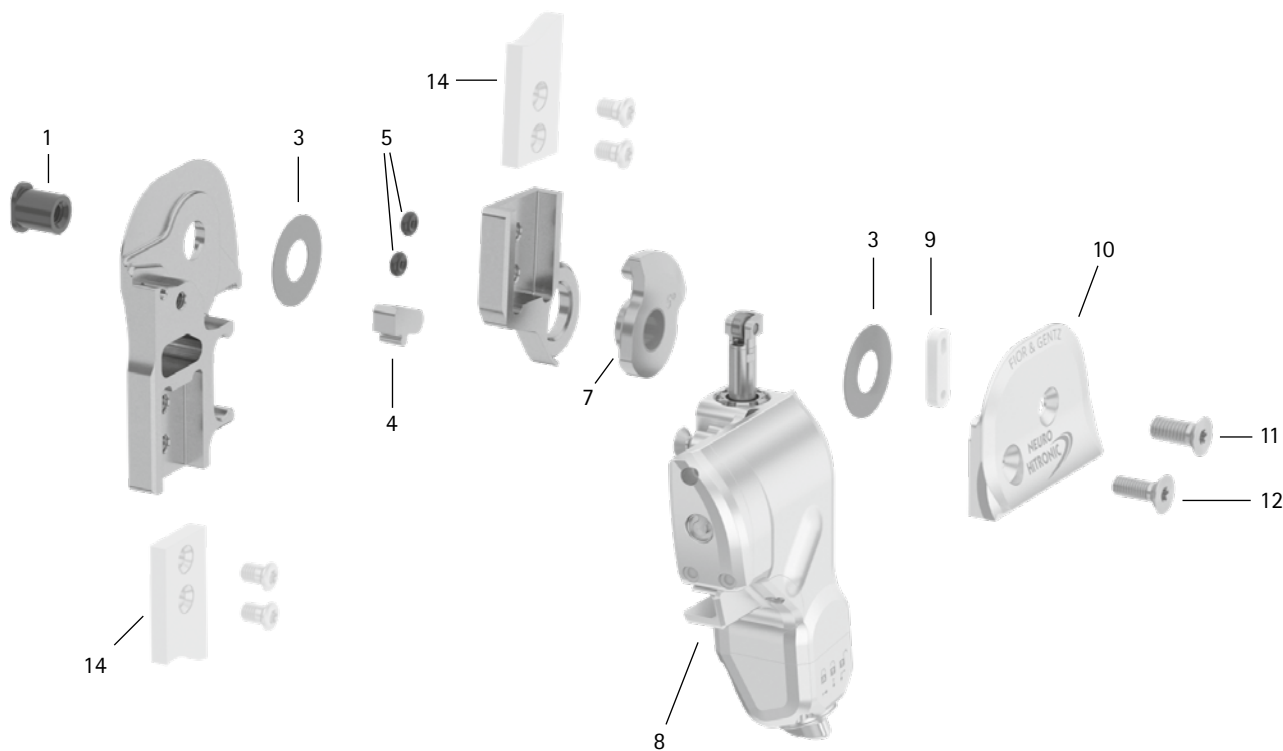
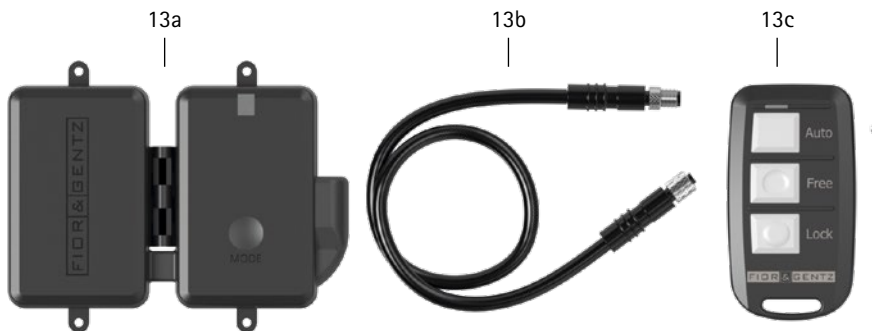
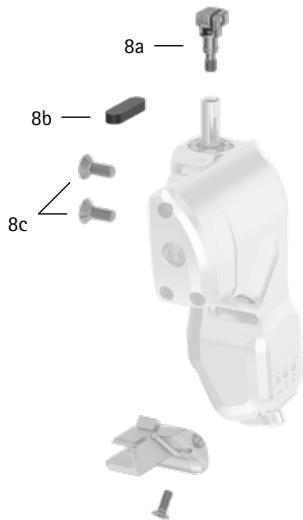


patient commission/configuration ID:

functional unit serial number:*

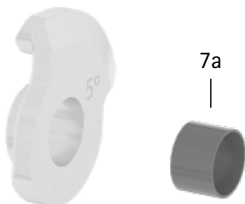

* You can find the serial number here:

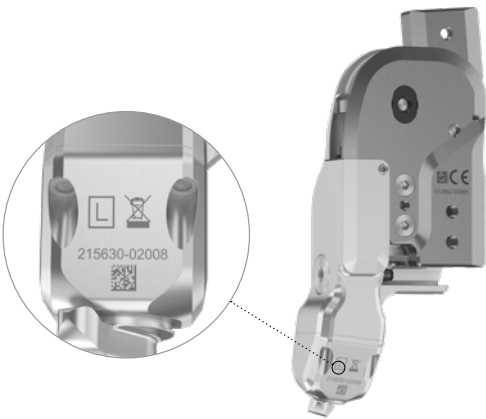

patient commission/configuration ID:

functional unit serial number:*



Item	20mm	Description	Potential Problem	Measure	Recommended Inspection, Potential Replacement**	Latest Replacement after
System Joint						
1	SB1069-L1110	bearing nut	wear	replacing	every 6 months	36 months
3	GS2411-040	sliding washer, Ø = 24mm	wear	replacing	every 6 months	18 months
3	GS2411-045					
3	GS2411-050					
3	GS2411-055					
3	GS2411-060					
4	SL9305-E000	0° extension stop	wear	replacing	every 6 months	if required
4	SL9305-E005	5° extension stop				
4	SL9305-E010	10° extension stop				
5	VE3771-012/26	O-ring damper	wear	replacing	every 6 months	18 months
7	SL0365-L	0° flexion stop disc with sliding bushing	wear	replacing	every 6 months	12 months
7	SL0365-R					
7	SL0365-2L	5° flexion stop disc with sliding bushing				
7	SL0365-2R					
7	SL0365-4L	10° flexion stop disc with sliding bushing				
7	SL0365-4R					
7a	BP1211-L077	sliding bushing	wear	replacing	every 6 months	18 months
8	SL3955-L	functional unit, left lateral	loss of function, wear	replacing	every 6 months	36 months
8	SL3955-R	functional unit, right lateral				
8a	SL0355-11	roll unit***	wear	replacing	every 6 months	6 months
8b	SL0355-12	air filter***	soiling	replacing	every 6 months	6 months
8c	SC1404-L10	countersunk flat head screw with hexalobular socket***	wear	replacing	every 6 months	36 months
9	SL0355-01	guide piece	wear	replacing	every 6 months	6 months
10	SL0355-L/AL	cover plate, left lateral	wear	replacing	every 6 months	36 months
10	SL0355-R/AL	cover plate, right lateral				
11	SC1406-L14	countersunk flat head screw with hexalobular socket (axle screw)	wear	replacing	every 6 months	36 months
12	SC1405-L14	countersunk flat head screw with hexalobular socket				
Controller						
13a	ET3850	controller with Bluetooth with lithium-polymer battery	outdated software	updating software	every 6 months	36 months
			low battery health	replacing		
13b	ET0711-01	connection cable for functional unit, 300mm	damage	replacing	every 6 months	if required
Remote Control						
13c	ET3840-P	remote control with Bluetooth for the patient	outdated software	updating software	every 6 months	36 months
Software for Mobile Devices						
-	-	software for mobile devices (operating system, Expert App, User App)	security gaps in the software	updating software	every 6 months	if required
System Anchors and System Side Bars						
14	see catalogue	system anchor and system side bar	wear or breakage	replacing	every 6 months	if required

** depending on the assessment of the distributor of the custom-made product regarding the patient's usage behaviour

*** is part of the functional unit