

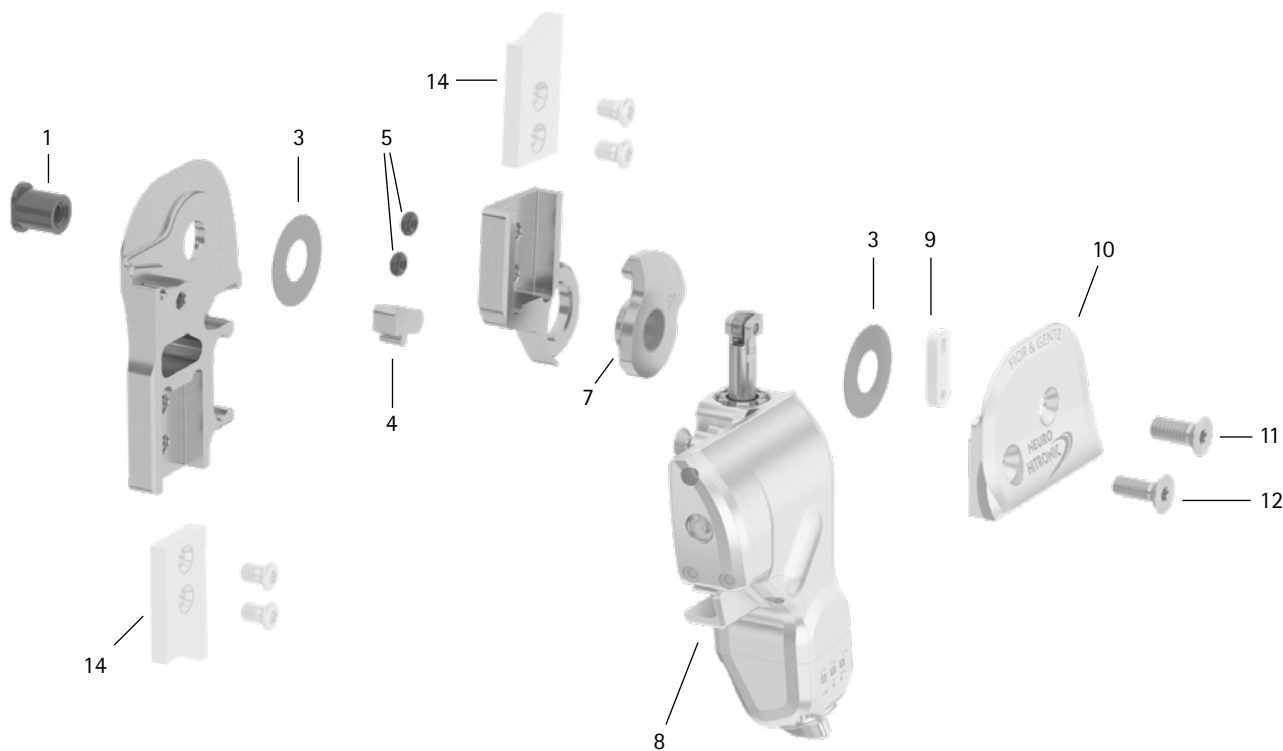
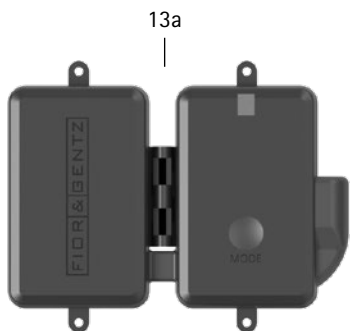
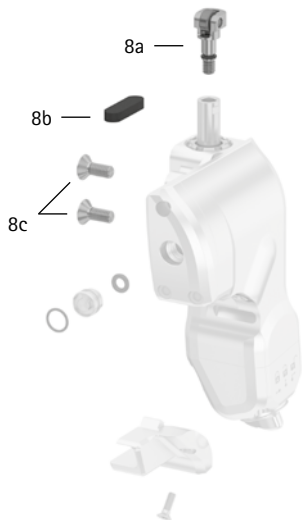


patient commission/configuration ID: .....

functional unit serial number:\* .....

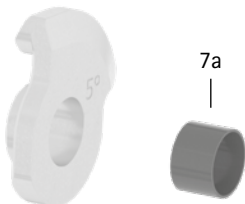

\* You can find the serial number here:

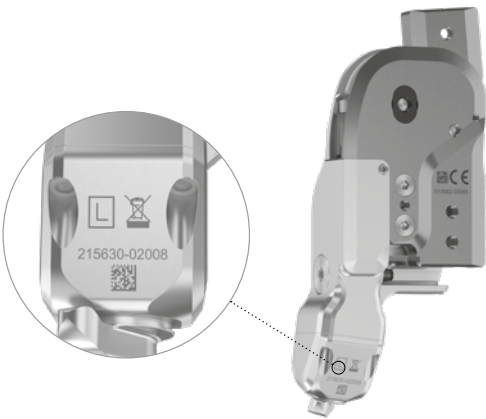

patient commission/configuration ID: .....

functional unit serial number:\* .....



| Item                                | 20mm          | Description                                                          | Potential Problem             | Measure                                  | Recommended Inspection, Potential Replacement** | Latest Replacement after |
|-------------------------------------|---------------|----------------------------------------------------------------------|-------------------------------|------------------------------------------|-------------------------------------------------|--------------------------|
| System Joint                        |               |                                                                      |                               |                                          |                                                 |                          |
| 1                                   | SB1069-L1110  | bearing nut                                                          | wear                          | replacing                                | every 6 months                                  | 36 months                |
| 3                                   | GS2411-040    | sliding washer, Ø = 24mm                                             | wear                          | replacing                                | every 6 months                                  | 18 months                |
| 3                                   | GS2411-045    |                                                                      |                               |                                          |                                                 |                          |
| 3                                   | GS2411-050    |                                                                      |                               |                                          |                                                 |                          |
| 3                                   | GS2411-055    |                                                                      |                               |                                          |                                                 |                          |
| 3                                   | GS2411-060    |                                                                      |                               |                                          |                                                 |                          |
| 4                                   | SL9305-E000   | 0° extension stop                                                    | wear                          | replacing                                | every 6 months                                  | if required              |
| 4                                   | SL9305-E005   | 5° extension stop                                                    |                               |                                          |                                                 |                          |
| 4                                   | SL9305-E010   | 10° extension stop                                                   |                               |                                          |                                                 |                          |
| 5                                   | VE3771-012/26 | O-ring damper                                                        | wear                          | replacing                                | every 6 months                                  | 18 months                |
| 7                                   | SL0365-L      | 0° flexion stop disc with sliding bushing                            | wear                          | replacing                                | every 6 months                                  | 12 months                |
| 7                                   | SL0365-R      |                                                                      |                               |                                          |                                                 |                          |
| 7                                   | SL0365-2L     | 5° flexion stop disc with sliding bushing                            |                               |                                          |                                                 |                          |
| 7                                   | SL0365-2R     |                                                                      |                               |                                          |                                                 |                          |
| 7                                   | SL0365-4L     | 10° flexion stop disc with sliding bushing                           |                               |                                          |                                                 |                          |
| 7                                   | SL0365-4R     |                                                                      |                               |                                          |                                                 |                          |
| 7a                                  | BP1211-L077   | sliding bushing                                                      | wear                          | replacing                                | every 6 months                                  | 18 months                |
| 8                                   | SL3955-L      | functional unit, left lateral                                        | loss of function, wear        | replacing                                | every 6 months                                  | 36 months                |
| 8                                   | SL3955-R      | functional unit, right lateral                                       |                               |                                          |                                                 |                          |
| 8a                                  | SL0355-11     | roll unit***                                                         | wear                          | replacing                                | every 6 months                                  | 6 months                 |
| 8b                                  | SL0355-12     | air filter***                                                        | soiling                       | replacing                                | every 6 months                                  | 6 months                 |
| 8c                                  | SC1404-L10    | countersunk flat head screw with hexalobular socket***               | wear                          | replacing                                | every 6 months                                  | 36 months                |
| -                                   | -             | pressure of the pneumatic spring****                                 | slow plunger movement         | inflate pressure of the pneumatic spring | every 6 months                                  | 6 months                 |
| 9                                   | SL0355-01     | guide piece                                                          | wear                          | replacing                                | every 6 months                                  | 6 months                 |
| 10                                  | SL0355-L/AL   | cover plate, left lateral                                            | wear                          | replacing                                | every 6 months                                  | 36 months                |
| 10                                  | SL0355-R/AL   | cover plate, right lateral                                           |                               |                                          |                                                 |                          |
| 11                                  | SC1406-L14    | countersunk flat head screw with hexalobular socket (axle screw)     | wear                          | replacing                                | every 6 months                                  | 36 months                |
| 12                                  | SC1405-L14    | countersunk flat head screw with hexalobular socket                  |                               |                                          |                                                 |                          |
| Controller                          |               |                                                                      |                               |                                          |                                                 |                          |
| 13a                                 | ET3860        | controller with Bluetooth®***** with lithium-polymer battery         | outdated software             | updating software                        | every 6 months                                  | 36 months                |
|                                     |               |                                                                      | low battery health            | replacing                                |                                                 |                          |
| 13b                                 | ET0711-01     | connection cable for functional unit, 300mm                          | damage                        | replacing                                | every 6 months                                  | if required              |
| Remote Control                      |               |                                                                      |                               |                                          |                                                 |                          |
| 13c                                 | ET3840-P      | remote control with Bluetooth® for the patient                       | outdated software             | updating software                        | every 6 months                                  | 36 months                |
| Software for Mobile Devices         |               |                                                                      |                               |                                          |                                                 |                          |
| -                                   | -             | software for mobile devices (operating system, Expert App, User App) | security gaps in the software | updating software                        | every 6 months                                  | if required              |
| System Anchors and System Side Bars |               |                                                                      |                               |                                          |                                                 |                          |
| 14                                  | see catalogue | system anchor and system side bar                                    | wear or breakage              | replacing                                | every 6 months                                  | if required              |

\*\* depending on the assessment of the distributor of the custom-made product regarding the patient's usage behaviour

\*\*\* is part of the functional unit

\*\*\*\* Also check the pressure of the pneumatic spring before handing over the orthosis and, if necessary, reinflate it.

\*\*\*\*\* The Bluetooth® word mark and logos are registered trademarks of Bluetooth SIG, Inc. and any use of such marks by FIOR & GENTZ is under license.