


patient commission/configuration ID \_\_\_\_\_

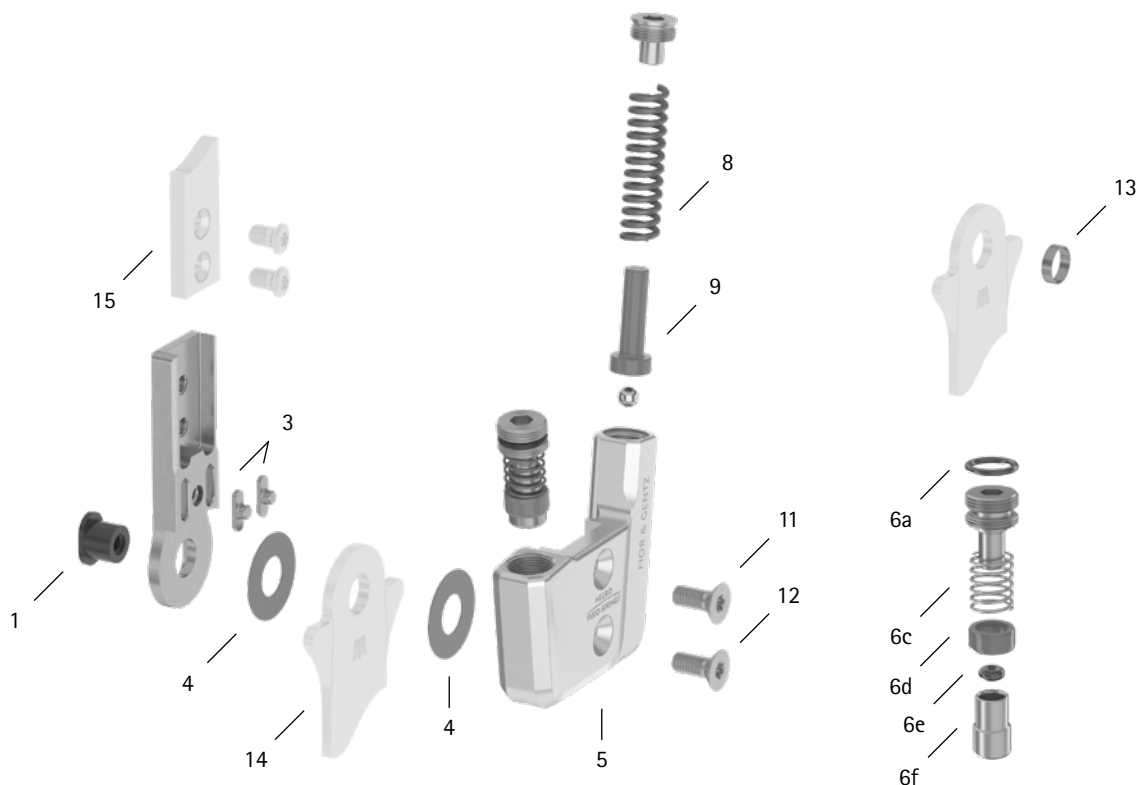

| Item                                                 | 10mm          | Description                                                           | Potential Problem      | Measure   | Recommended Inspection, Potential Replacement* | Latest Replacement after |
|------------------------------------------------------|---------------|-----------------------------------------------------------------------|------------------------|-----------|------------------------------------------------|--------------------------|
| System Joint                                         |               |                                                                       |                        |           |                                                |                          |
| 1                                                    | SB6039-L0580  | bearing nut                                                           | wear                   | replacing | every 6 months                                 | 36 months                |
| 3                                                    | SF0390-01     | feather key with pin                                                  | breakage               | replacing | every 6 months                                 | 36 months                |
| 4                                                    | GS1206-040    | sliding washer, Ø = 12mm                                              | wear                   | replacing | every 6 months                                 | 18 months                |
| 4                                                    | GS1206-045    |                                                                       |                        |           |                                                |                          |
| 4                                                    | GS1206-050    |                                                                       |                        |           |                                                |                          |
| 4                                                    | GS1206-055    |                                                                       |                        |           |                                                |                          |
| 4                                                    | GS1206-060    |                                                                       |                        |           |                                                |                          |
| 5                                                    | SF0860-L/AL   | cover plate, left lateral or right medial**                           | wear                   | replacing | every 6 months                                 | 36 months                |
| 5                                                    | SF0860-R/AL   | cover plate, left medial or right lateral**                           |                        |           |                                                |                          |
| 6a                                                   | VE3771-04/10  | O-ring for securing the alignment screw**                             | wear                   | replacing | every 6 months                                 | 18 months                |
| 6c                                                   | FE1615-01     | coil spring**                                                         | wear                   | replacing | every 6 months                                 | 18 months                |
| 6d                                                   | GS0604-350    | sliding bushing**                                                     | wear                   | replacing | every 6 months                                 | 18 months                |
| 6e                                                   | PN0003-L02    | stop damper**                                                         | wear                   | replacing | every 6 months                                 | 18 months                |
| 6f                                                   | SH0490-01     | plunger**                                                             | wear                   | replacing | every 6 months                                 | 36 months                |
| 8                                                    | FE1622-01     | coil spring, golden**                                                 | wear                   | replacing | every 6 months                                 | 18 months                |
| 9                                                    | SF0840-07     | pin**                                                                 | wear                   | replacing | every 6 months                                 | 36 months                |
| 11                                                   | SC1403-L08    | countersunk flat head screw with hexalobular socket**                 | wear                   | replacing | every 6 months                                 | 36 months                |
| 12                                                   | SC1403-L08    | countersunk flat head screw with hexalobular socket (axle screw)**    |                        |           |                                                |                          |
| 5-12                                                 | SF8970-L/AL   | functional unit plug + go modularity, left lateral or right medial*** | loss of function, wear | replacing | every 6 months                                 | 36 months                |
| 5-12                                                 | SF8970-R/AL   | functional unit plug + go modularity, left medial or right lateral*** |                        |           |                                                |                          |
| System Stirrups, System Anchors and System Side Bars |               |                                                                       |                        |           |                                                |                          |
| 13                                                   | BP0706-L020   | sliding bushing                                                       | wear                   | replacing | every 6 months                                 | 18 months                |
| 14                                                   | see catalogue | system stirrup                                                        | wear or breakage       | replacing | every 6 months                                 | 48 months                |
| 15                                                   | see catalogue | system anchor and system side bar                                     | wear or breakage       | replacing | every 6 months                                 | if required              |

\* depending on the assessment of the distributor of the custom-made product regarding the patient's usage behaviour

\*\* is part of the functional unit

\*\*\* included system components can be exchanged separately