


patient commission/configuration ID _____

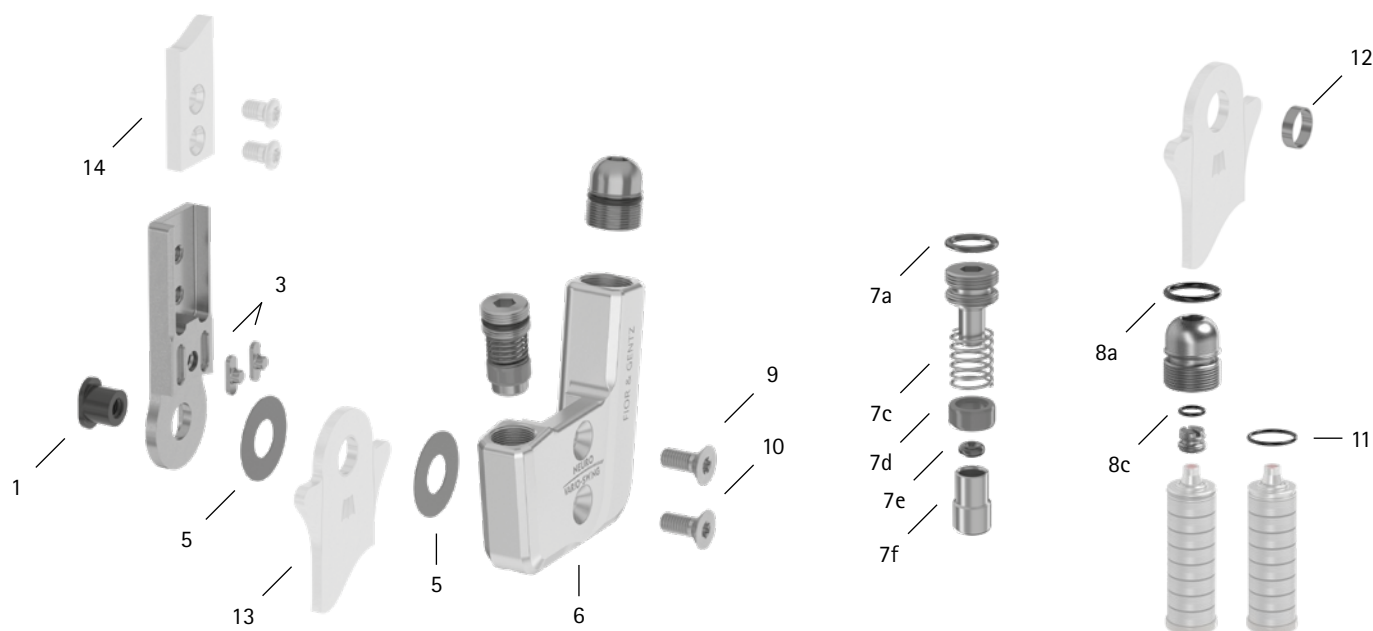

Item	14mm	Description	Potential Problem	Measure	Recommended Inspection, Potential Replacement*	Latest Replacement after
System Joint						
1	SB8559-L0630	bearing nut	wear	replacing	every 6 months	36 months
3	SF0392-02	feather key with pin	breakage	replacing	every 6 months	36 months
5	GS2009-040	sliding washer, Ø = 20mm	wear	replacing	every 6 months	18 months
5	GS2009-045					
5	GS2009-050					
5	GS2009-055					
5	GS2009-060					
6	SH0462-L/AL	cover plate, left lateral or right medial**	wear	replacing	every 6 months	36 months
6	SH0462-R/AL	cover plate, left medial or right lateral**				
7a	VE3771-08/15	O-ring for securing the alignment screw**	wear	replacing	every 6 months	18 months
7c	FE1916-01	coil spring**	wear	replacing	every 6 months	18 months
7d	GS1007-350	sliding bushing**	wear	replacing	every 6 months	18 months
7e	VE3771-010/20	O-ring damper**	wear	replacing	every 6 months	18 months
7f	SH0492-01	plunger**	wear	replacing	every 6 months	36 months
8a	VE3771-09/15	O-ring for securing the alignment screw**	wear	replacing	every 6 months	18 months
8c	VE3771-04/10	O-ring for securing the motion limiting screw**	wear	replacing	every 6 months	18 months
9	SC1405-L11	countersunk flat head screw with hexalobular socket**	wear	replacing	every 6 months	36 months
10	SC1405-L11	countersunk flat head screw with hexalobular socket (axle screw)**				
6-10	SH4972-L/AL	functional unit plug + go modularity, left lateral or right medial***	loss of function, wear	replacing	every 6 months	36 months
6-10	SH4972-R/AL	functional unit plug + go modularity, left medial or right lateral***				

patient commission/configuration ID _____



Item	14mm	Description	Potential Problem	Measure	Recommended Inspection, Potential Replacement*	Latest Replacement after
Spring Units						
11	VE3771-07/10	O-ring for securing the spring unit	wear	replacing	every 6 months	18 months
-	SF5802-15/05	spring unit, blue, normal, max. 15° range of motion	wear	replacing	every 6 months	18 months
-	SF5802-15/11	spring unit, green, medium, max. 15° range of motion	radial move of disc springs	realigning disc springs with pliers		
-	SF5802-09/16	spring unit, white, strong, max. 10° range of motion				
-	SF5802-10/29	spring unit, yellow, very strong, max. 10° range of motion				
-	SF5802-05/53	spring unit, red, extra strong, max. 5° range of motion	noise of spring unit	greasing spring unit with spray oil (article no. FT3000-15)		
System Stirrups, System Anchors and System Side Bars						
12	BR1009-L025	sliding bushing	wear	replacing	every 6 months	18 months
13	see catalogue	system stirrup	wear or breakage	replacing	every 6 months	48 months
14	see catalogue	system anchor and system side bar	wear or breakage	replacing	every 6 months	if required

* depending on the assessment of the distributor of the custom-made product regarding the patient's usage behaviour
** is part of the functional unit
*** included system components can be exchanged separately