


patient commission/configuration ID _____

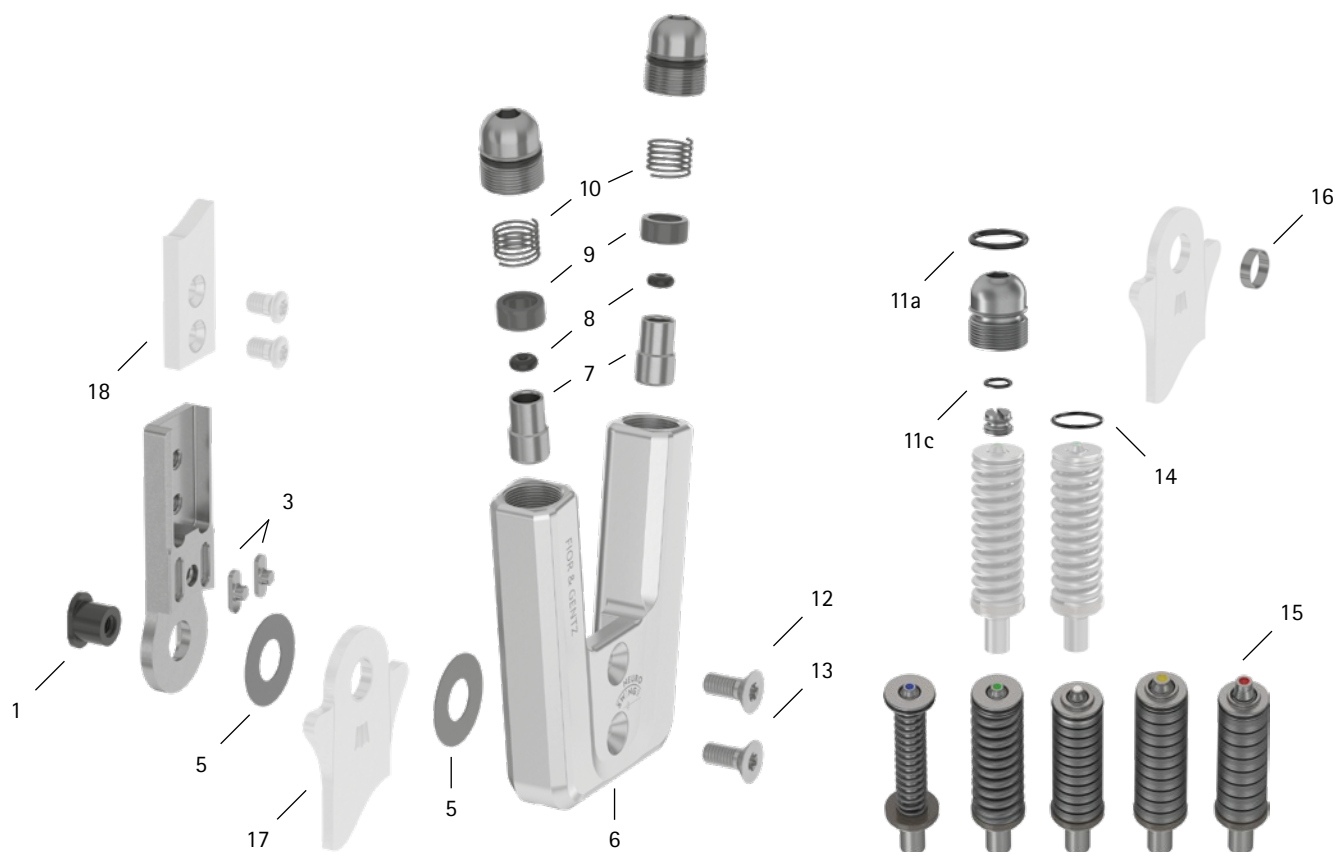

Item	14mm	Description	Potential Problem	Measure	Recommended Inspection, Potential Replacement*	Latest Replacement after
System Joint						
1	SB8559-L0630	bearing nut	wear	replacing	every 6 months	36 months
3	SF0392-02	feather key with pin	breakage	replacing	every 6 months	36 months
5	GS2009-040	sliding washer, Ø = 20mm	wear	replacing	every 6 months	18 months
5	GS2009-045					
5	GS2009-050					
5	GS2009-055					
5	GS2009-060					
6	SH0562-AL	cover plate**	wear	replacing	every 6 months	36 months
7	SH0492-01	plunger**	wear	replacing	every 6 months	36 months
8	VE3771-010/20	O-ring damper**	wear	replacing	every 6 months	18 months
9	GS1007-350	sliding bushing**	wear	replacing	every 6 months	18 months
10	FE1916-01	coil spring**	wear	replacing	every 6 months	18 months
11a	VE3771-09/15	O-ring for securing the alignment screw**	wear	replacing	every 6 months	18 months
11c	VE3771-04/10	O-ring for securing the motion limiting screw**	wear	replacing	every 6 months	18 months
12	SC1405-L11	countersunk flat head screw with hexalobular socket**	wear	replacing	every 6 months	36 months
13	SC1405-L11	countersunk flat head screw with hexalobular socket (axle screw)**				
6-13	SH5972-AL	functional unit plug + go modularity***	loss of function or wear	replacing	every 6 months	36 months

patient commission/configuration ID _____



Item	14mm	Description	Potential Problem	Measure	Recommended Inspection, Potential Replacement*	Latest Replacement after
Spring Units						
14	VE3771-07/10	O-ring for securing the spring unit	wear	replacing	every 6 months	18 months
15	SH5802-15/05	spring unit, blue, normal, max. 15° range of motion	wear	replacing	every 6 months	18 months
15	SH5802-15/11	spring unit, green, medium, max. 15° range of motion	radial move of disc springs	realigning disc springs with pliers		
15	SH5802-09/16	spring unit, white, strong, max. 10° range of motion				
15	SH5802-10/29	spring unit, yellow, very strong, max. 10° range of motion				
15	SH5802-05/53	spring unit, red, extra strong, max. 5° range of motion	noise of spring unit	greasing spring unit with spray oil (article no. FT3000-15)		
System Stirrups, System Anchors and System Side Bars						
16	BR1009-L025	sliding bushing	wear	replacing	every 6 months	18 months
17	see catalogue	system stirrup	wear or breakage	replacing	every 6 months	48 months
18	see catalogue	system anchor and system side bar	wear or breakage	replacing	every 6 months	if required

* depending on the assessment of the distributor of the custom-made product regarding the patient's usage behaviour
** is part of the functional unit
*** included system components can be exchanged separately