


patient commission/configuration ID \_\_\_\_\_

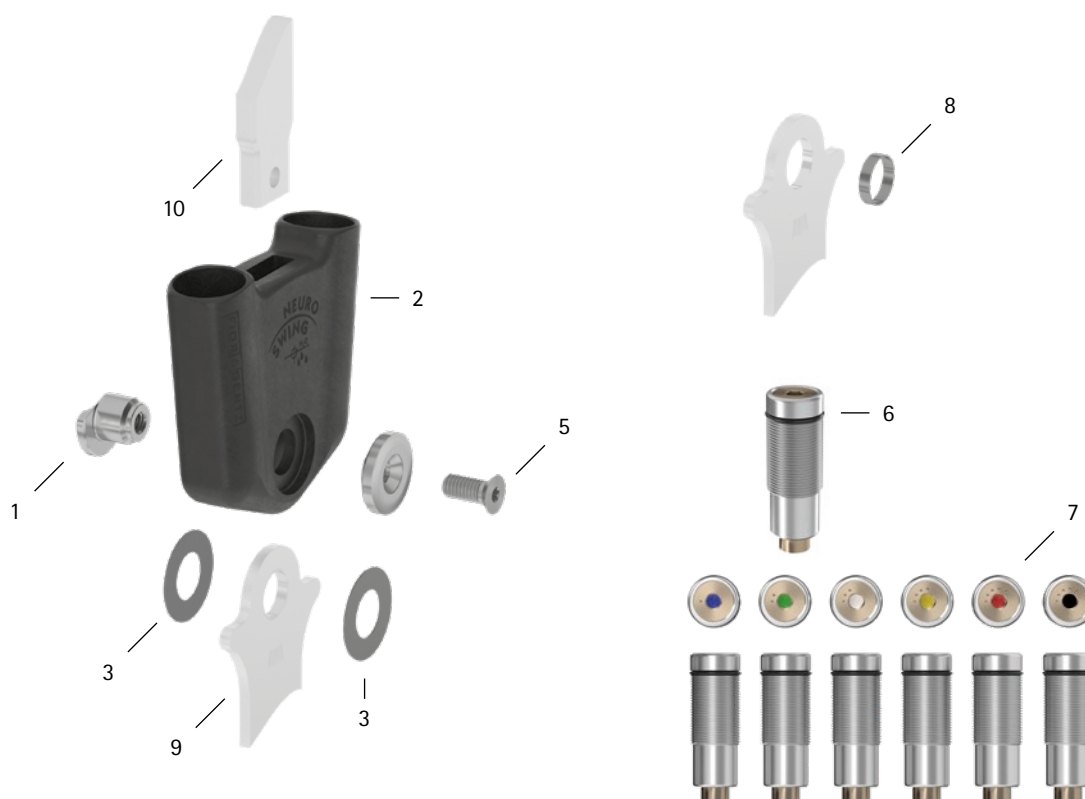

| Item                                                        | 16mm           | Description                                                      | Potential Problem                                 | Measure                              | Recommended Inspection, Potential Replacement* | Latest Replacement after |
|-------------------------------------------------------------|----------------|------------------------------------------------------------------|---------------------------------------------------|--------------------------------------|------------------------------------------------|--------------------------|
| <b>System Joint</b>                                         |                |                                                                  |                                                   |                                      |                                                |                          |
| 1                                                           | SF0593-C/1     | bearing nut                                                      | wear or breakage                                  | replacing                            | every 6 months                                 | 48 months                |
| 2                                                           | SF0503-C       | joint case                                                       | wear or breakage                                  | replacing                            | every 6 months                                 | 48 months                |
| 3                                                           | GS2413-040     | sliding washer, Ø = 24mm                                         | wear                                              | replacing                            | every 6 months                                 | 18 months                |
| 3                                                           | GS2413-045     |                                                                  |                                                   |                                      |                                                |                          |
| 3                                                           | GS2413-050     |                                                                  |                                                   |                                      |                                                |                          |
| 3                                                           | GS2413-055     |                                                                  |                                                   |                                      |                                                |                          |
| 3                                                           | GS2413-060     |                                                                  |                                                   |                                      |                                                |                          |
| 5                                                           | SC1406-L14     | countersunk flat head screw with hexalobular socket              | wear                                              | replacing                            | every 6 months                                 | 36 months                |
| <b>Spring Units and Static Dorsiflexion Stop</b>            |                |                                                                  |                                                   |                                      |                                                |                          |
| 6                                                           | VE3771-12/12   | O-ring for securing the spring unit and static dorsiflexion stop | wear                                              | replacing                            | every 6 months                                 | 18 months                |
| 7                                                           | SF5803-C/15/07 | spring unit, blue, normal, max. 15° range of motion              | noise of spring unit/<br>static dorsiflexion stop | replacing                            | every 6 months                                 | 18 months                |
| 7                                                           | SF5803-C/15/15 | spring unit, green, medium, max. 15° range of motion             |                                                   |                                      |                                                |                          |
| 7                                                           | SF5803-C/10/21 | spring unit, white, strong, max. 10° range of motion             |                                                   |                                      |                                                |                          |
| 7                                                           | SF5803-C/10/31 | spring unit, yellow, very strong, max. 10° range of motion       |                                                   |                                      |                                                |                          |
| 7                                                           | SF5803-C/05/63 | spring unit, red, extra strong, max. 5° range of motion          |                                                   |                                      |                                                |                          |
| 7                                                           | SF5803-C/0     | static dorsiflexion stop, black, 0° range of motion              |                                                   |                                      |                                                |                          |
| <b>System Stirrups, System Anchors and System Side Bars</b> |                |                                                                  |                                                   |                                      |                                                |                          |
| 8                                                           | BR1312-L030    | sliding bushing                                                  | wear                                              | replacing                            | every 6 months                                 | 18 months                |
| 9                                                           | see catalogue  | system stirrup                                                   | wear or breakage                                  | replacing                            | every 6 months                                 | 48 months                |
| 10                                                          | see catalogue  | system anchor and system side bar                                | wear or breakage                                  | replacing                            | every 6 months                                 | if required              |
| -                                                           | -              | adhesive bond (system side bar/system anchor and system joint)   | breakage                                          | adhering new carbon system component | every 6 months                                 | if required              |

\* depending on the assessment of the distributor of the custom-made product regarding the patient's usage behaviour

WP0067-GB-2025-09