


patient commission/configuration ID _____

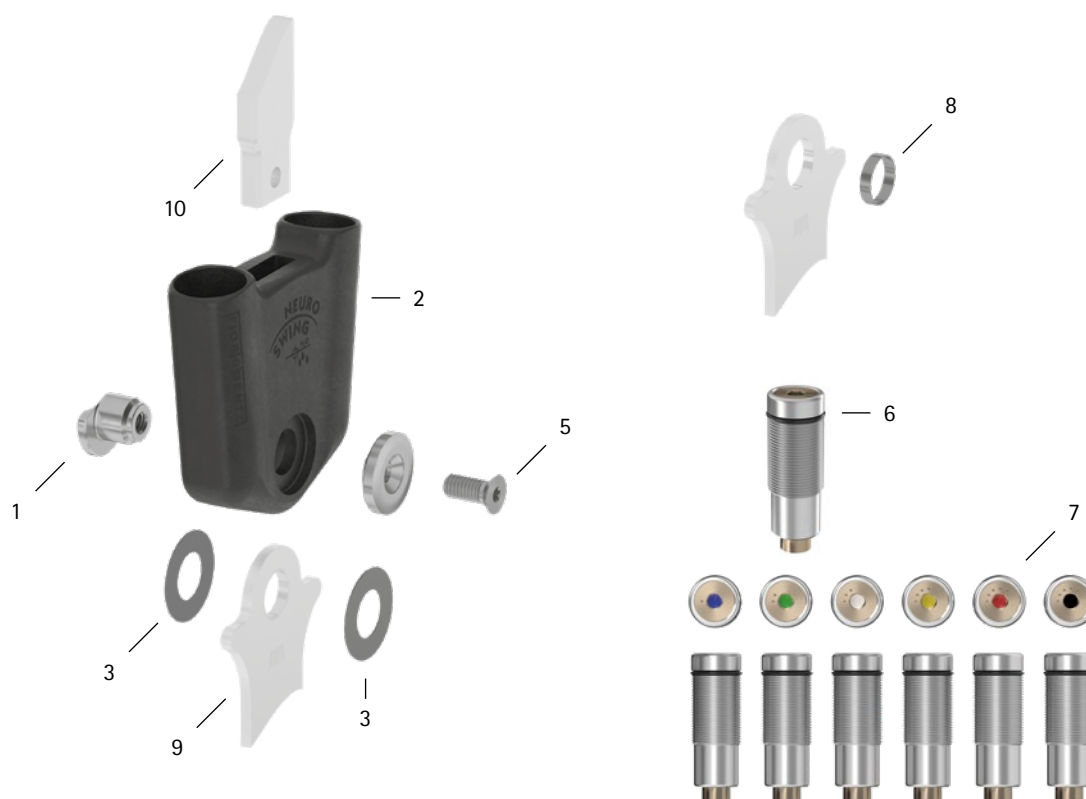

Item	20mm	Description	Potential Problem	Measure	Recommended Inspection, Potential Replacement*	Latest Replacement after
System Joint						
1	SF0595-C/1	bearing nut	wear or breakage	replacing	every 6 months	48 months
2	SF0505-C	joint case	wear or breakage	replacing	every 6 months	48 months
3	GS2815-040	sliding washer, Ø = 28mm	wear	replacing	every 6 months	18 months
3	GS2815-045					
3	GS2815-050					
3	GS2815-055					
3	GS2815-060					
5	SC1406-L14	countersunk flat head screw with hexalobular socket	wear	replacing	every 6 months	36 months
Spring Units and Static Dorsiflexion Stop						
6	VE3771-140/15	O-ring for securing the spring unit and static dorsiflexion stop	wear	replacing	every 6 months	18 months
7	SF5805-C/15/18	spring unit, blue, normal, max. 15° range of motion	noise of spring unit/ static dorsiflexion stop	replacing	every 6 months	18 months
7	SF5805-C/15/25	spring unit, green, medium, max. 15° range of motion				
7	SF5805-C/10/40	spring unit, white, strong, max. 10° range of motion				
7	SF5805-C/10/60	spring unit, yellow, very strong, max. 10° range of motion				
7	SF5805-C/05/99	spring unit, red, extra strong, max. 5° range of motion				
7	SF5805-C/0	static dorsiflexion stop, black, 0° range of motion				
System Stirrups, System Anchors and System Side Bars						
8	BR1514-L030	sliding bushing	wear	replacing	every 6 months	18 months
9	see catalogue	system stirrup	wear or breakage	replacing	every 6 months	48 months
10	see catalogue	system anchor and system side bar	wear or breakage	replacing	every 6 months	if required
-	-	adhesive bond (system side bar/system anchor and system joint)	breakage	adhering new carbon system component	every 6 months	if required

* depending on the assessment of the distributor of the custom-made product regarding the patient's usage behaviour

WP0068-GB-2025-09