


patient commission/configuration ID _____

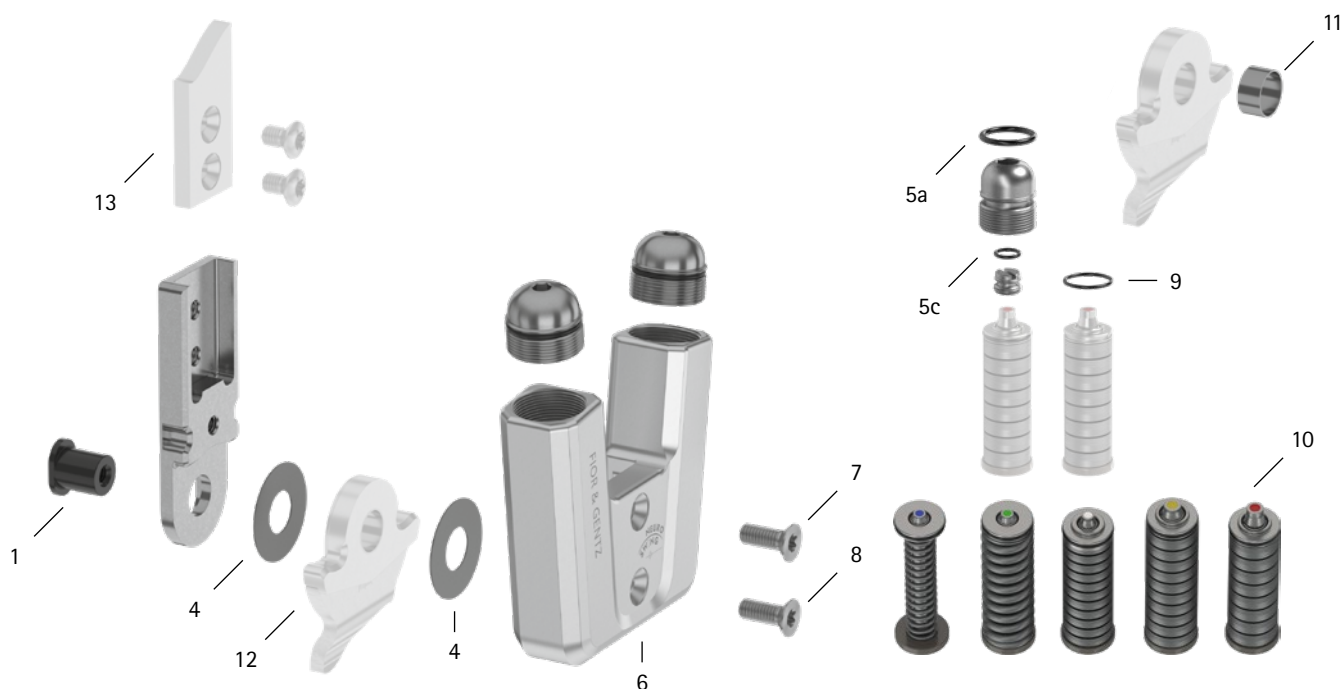

Item	24mm	Description	Potential Problem	Measure	Recommended Inspection, Potential Replacement*	Latest Replacement after
System Joint						
1	SB1262-L1400	bearing nut	wear	replacing	every 6 months	36 months
4	GS3013-040	sliding washer, Ø = 30mm	wear	replacing	every 6 months	18 months
4	GS3013-045					
4	GS3013-050					
4	GS3013-055					
4	GS3013-060					
5a	VE3771-160/15	O-ring for securing the alignment screw**	wear	replacing	every 6 months	18 months
5c	VE3771-07/10	O-ring for securing the motion limiting screw**	wear	replacing	every 6 months	18 months
6	SF0567-2/AL	cover plate**	wear	replacing	every 6 months	36 months
7	SC1416-L16	countersunk flat head screw with hexalobular socket**	wear	replacing	every 6 months	36 months
8	SC1416-L16	countersunk flat head screw with hexalobular socket (axle screw)**				
5-8	SF5977-AL	functional unit***	loss of function, wear	replacing	every 6 months	36 months
Spring Units						
9	VE3771-140/10	O-ring for securing the spring unit	wear	replacing	every 6 months	18 months
10	SF5807-15/12	spring unit, blue, normal, max. 15° range of motion	wear	replacing	every 6 months	18 months
10	SF5807-15/25	spring unit, green, medium, max. 15° range of motion				
10	SF5807-10/53	spring unit, white, strong, max. 10° range of motion	radial move of disc springs	realigning disc springs with pliers		
10	SF5807-10/80	spring unit, yellow, very strong, max. 10° range of motion	noise of spring unit	greasing spring unit with spray oil (article no. FT3000-15)		
10	SF5807-05/99	spring unit, red, extra strong, max. 5° range of motion				
System Stirrups, System Anchors and System Side Bars						
11	BR1413-L070	sliding bushing	wear	replacing	every 6 months	18 months
12	see catalogue	system stirrup	wear or breakage	replacing	every 6 months	48 months
13	see catalogue	system anchor and system side bar	wear or breakage	replacing	every 6 months	if required

* depending on the assessment of the distributor of the custom-made product regarding the patient's usage behaviour

** is part of the functional unit

*** included system components can be exchanged separately

WP0127-GB-2024-12