



patient commission/configuration ID _____

functional unit serial number* _____

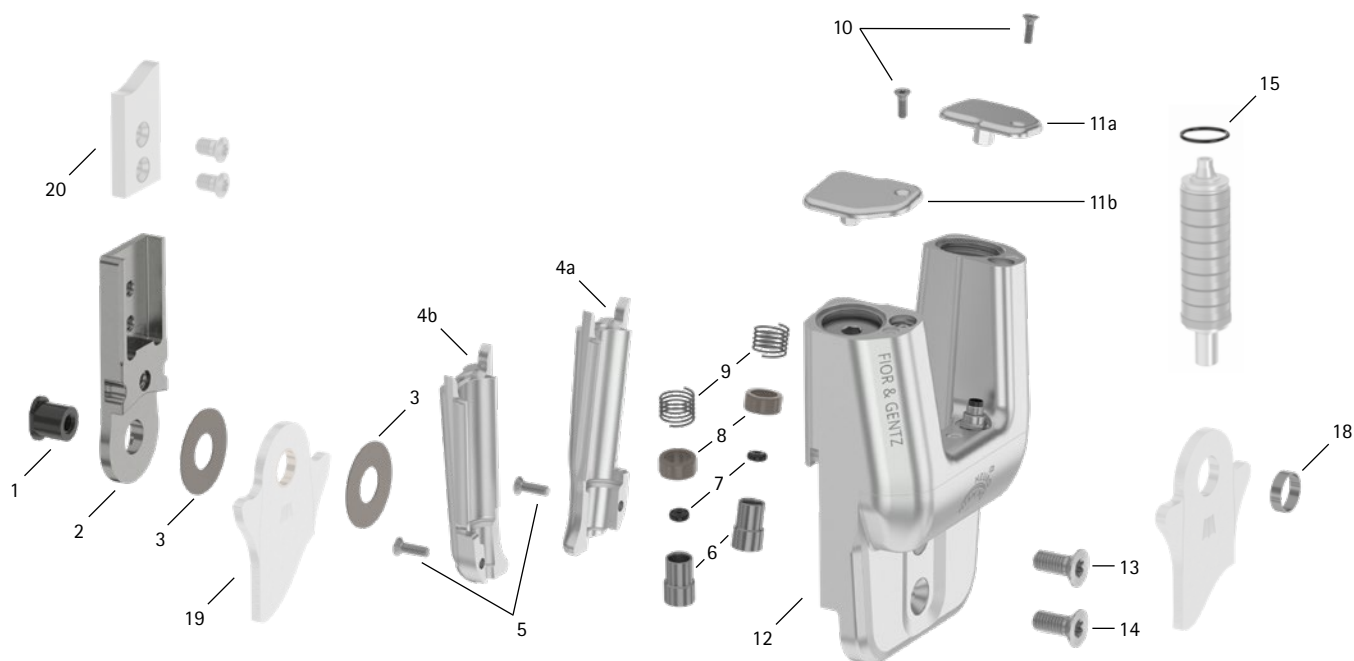

* You can find the serial number here:

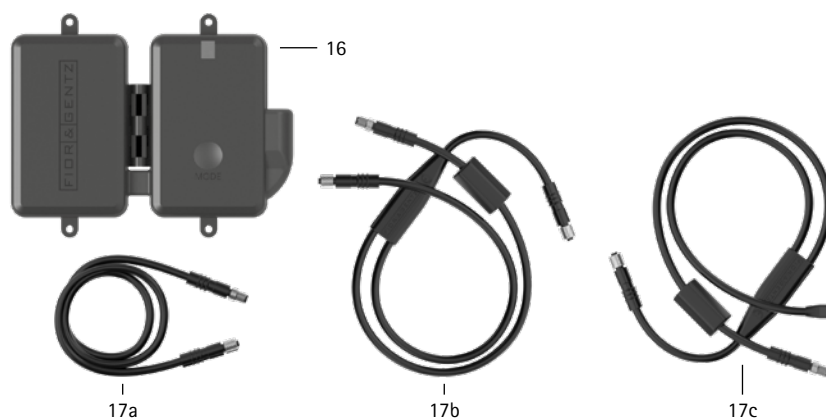

Item	20mm	Description	Potential Problem	Measure	Recommended Inspection, Potential Replacement**	Latest Replacement after
System Joint						
1	SB1069-L0960	bearing nut	wear	replacing	every 6 months	36 months
3	GS2611-040	sliding washer, Ø = 26mm	wear	replacing	every 6 months	18 months
3	GS2611-045					
3	GS2611-050					
3	GS2611-055					
3	GS2611-060					
5	SC1403-L10	countersunk flat head screw with hexalobular socket	wear	replacing	every 6 months	36 months
6	SH0493-01	plunger***	wear	replacing	every 6 months	36 months
7	VE3771-012/26	O-ring damper***	wear	replacing	every 6 months	18 months
8	GS1108-500	sliding bushing***	wear	replacing	every 6 months	18 months
9	FE1027-01	pressure spring***	wear	replacing	every 6 months	18 months
10	SC1403-L08/1	countersunk flat head screw with hexalobular socket	wear	replacing	every 6 months	36 months
13	SC1416-L14	countersunk flat head screw with hexalobular socket***	wear	replacing	every 6 months	36 months
14	SC1416-L14	countersunk flat head screw with hexalobular socket (axle screw)***				
4-14	SH8975-AL	functional unit	loss of function, wear	replacing	every 6 months	36 months

patient commission/configuration ID _____

functional unit serial number* _____



Item	20mm	Description	Potential Problem	Measure	Recommended Inspection, Potential Replacement**	Latest Replacement after
Spring Units						
15	VE3771-11/10	O-ring for securing the spring unit	wear	replacing	every 6 months	18 months
-	SH5805-15/18	spring unit, blue, normal, max. 15° range of motion	wear radial move of disc springs noise of spring unit	replacing	every 6 months	18 months
-	SH5805-15/25	spring unit, green, medium, max. 15° range of motion		realigning disc springs with pliers		
-	SH5805-10/40	spring unit, white, strong, max. 10° range of motion		greasing spring unit with spray oil (article no. FT3000-15)		
-	SH5805-10/60	spring unit, yellow, very strong, max. 10° range of motion				
-	SH5805-05/99	spring unit, red, extra strong, max. 5° range of motion				
Controller						
16	ET3860	controller with Bluetooth®**** with lithium-polymer battery	outdated software low battery health	updating software replacing	every 6 months	36 months
17a	ET0711-03	connection cable for functional unit NEURO HiSWING R+, 510mm	damage	replacing	every 6 months	if required
17b	ET0713-02	connection cable for functional unit NEURO HiTRONIC and NEURO HiSWING R+, 660mm	damage	replacing	every 6 months	if required
17c	ET0714-02	connection cable for functional unit NEURO TRONIC and NEURO HiSWING R+, 660mm	damage	replacing	every 6 months	if required
Software for Mobile Devices						
-	-	software for mobile devices (operating system, Expert App, User App)	security gaps in the software	updating software	every 6 months	if required
System Stirrups, System Anchors and System Side Bars						
18	BR1211-L030	sliding bushing	wear	replacing	every 6 months	18 months
19	see catalogue	system stirrup	wear or breakage	replacing	every 6 months	48 months
20	see catalogue	system anchor and system side bar	wear or breakage	replacing	every 6 months	if required

** depending on the assessment of the distributor of the custom-made product regarding the patient's usage behaviour

*** is part of the functional unit

**** The Bluetooth® word mark and logos are registered trademarks of Bluetooth SIG, Inc. and any use of such marks by FIOR & GENTZ is under license.