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Druckversion: 2020-05

Instructions for Use for Patients

Articulated Side Bars for Knee Orthoses

Dear Patient,

You have received from your orthotist or a qualified/trained expert an individually produced orthosis with the high quality FIOR & GENTZ articulated side bars for knee orthoses.

1. Safety Instructions

1.1 Classification of the Safety Instructions

 DANGER	Important information about a possible dangerous situation which, if not avoided, leads to death or irreversible injuries.
 WARNING	Important information about a possible dangerous situation which, if not avoided, leads to reversible injuries that need medical treatment.
 CAUTION	Important information about a possible dangerous situation which, if not avoided, leads to light injuries that do not need medical treatment.
NOTICE	Important information about a possible situation which, if not avoided, leads to damage of the product.

All serious incidents according to Regulation (EU) 2017/745 which are related to the product have to be reported to the manufacturer and to the competent authority of the Member State in which the orthotist or qualified/trained expert and/or the patient is established.

1.2 All Instructions for Your Safety

DANGER

Potential Traffic Accident Due to Limited Driving Ability

Gather information about all issues concerning safety and security and potential dangers before driving a motor vehicle with orthosis.

WARNING

Risk of Falling Due to Loosely Attached Cover Plate

Do not secure screws for the articulated side bar on your own. Consult your orthotist or a qualified/trained expert about the maintenance intervals to be respected.

WARNING

Risk of Falling Due to Improper Handling

Have your orthotist or a qualified/trained expert inform you about the correct use of the articulated side bar and potential dangers. Avoid contact with moisture and water.

⚠ WARNING

Risk of Falling Due to Orthosis Adjustments

If you notice any changes on the orthosis (e.g. loosely attached joint components), immediately contact your orthotist or a qualified/trained expert.

NOTICE

Limitation of Joint Function Due to Improper Cleaning

Clean the orthosis and the articulated side bar as described in these instructions for use. Do not grease the articulated side bar on your own. If necessary, consult your orthotist or a qualified/trained expert.

2. Application

The articulated side bars for knee orthoses are exclusively for use for orthotic fittings of the knee joint. They must be handled by an orthotist or qualified/trained expert. All FIOR & GENTZ joints were developed for everyday life activities such as standing and walking. Extreme loads connected to activities like running, climbing and parachuting are excluded.

3. Cleaning the Orthosis

Clean the orthosis regularly. Do not immerse the orthosis in water or other liquids. Instead, clean the orthosis with a damp cloth. Leave out the area around the joint and clean it only superficially with a dry cloth. Remove visible dust and lint from the mechanics by using tweezers. Check the orthosis in straight and flexed position.

4. Maintenance of the Orthosis

Ask your orthotist or a qualified/trained expert to check the joint of your orthosis every **6 months**. Bring these instructions for use to each follow-up and let your orthotist or a qualified/trained expert enter the next appointment in the table on the back. Never carry out maintenance work or other adjustments and repairs yourself. In the case of children and people with cognitive impairments, we would like to point out to you as parents or care team that you must regularly check the orthosis and the joint for signs of wear. If you notice any changes, immediately contact your orthotist or a qualified/trained expert.

5. Disposal of the Orthosis

If you no longer need the orthosis, please return it to your orthotist or a qualified/trained expert. The product must not be disposed of with the residual waste (fig. 1).

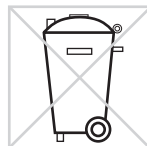
fig. 1

6. CE Conformity

We declare that our medical devices as well as our accessories for medical devices are in conformity with the requirements of Regulation (EU) 2017/745. Therefore, the FIOR & GENTZ products bear the CE marking.

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