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Patient Information for Therapeutic Shoes

Dear Patient,

You have received from your supplier a FIOR & GENTZ therapeutic shoe.

1. Safety Instructions

1.1 Classification of the Safety Instructions

DANGER

Important information about a possible dangerous situation which, if not avoided, leads to death or irreversible injuries.

WARNING

Important information about a possible dangerous situation which, if not avoided, leads to reversible injuries that need medical treatment.

CAUTION

Important information about a possible dangerous situation which, if not avoided, leads to light injuries that do not need medical treatment.

NOTICE

Important information about a possible situation which, if not avoided, leads to damage of the product.

All serious incidents connected to the product shall be reported to the manufacturer or your orthopaedic shoemaker.

.....

1.2 All Instructions for Your Safety

DANGER

Potential Traffic Accident Due to Limited Driving Ability

Gather information about all issues concerning safety and security as well as potential dangers before driving a motor vehicle with a therapeutic shoe.

WARNING

Risk of Falling Due to Improper Handling

Have your orthopaedic shoemaker inform you about the correct use of the shoe and potential dangers.

WARNING

Risk of Infection of Existing Wounds Due to Multiple Persons Wearing the Shoe

Only wear the therapeutic shoe yourself. For hygienic reasons, do not pass it on to third parties.

WARNING

Risk of Infections and Wounds for Patients with Diabetic Foot Syndrome

Have your orthopaedic shoemaker select a therapeutic shoe suitable for your risk group. Also note the following:

- Wear your therapeutic shoe with a custom-made diabetic foot orthotic to avoid chafing due to an incorrect load/relief.
- Wear closed therapeutic shoes or therapeutic shoes with protective caps to avoid the risk of infection from dirt entering.

.....

 **WARNING**

Jeopardising the Therapy Goal Due to the Use of a Relief Shoe Without Crutches

Use crutches when wearing a relief shoe in order to reduce the wear of the sole and thus prevent a loss of function of the shoe and the resulting insecurities in gait.

 **CAUTION**

Postural Damage Due to Different Heel Heights

Wear a height compensation shoe on the healthy foot, e.g. a matching FIOR & GENTZ TWINSHOE.

2. Application

All therapeutic shoes by FIOR & GENTZ are to be used exclusively for the therapeutic treatment of the foot. They offer ample space for bandages and can be opened very wide so that they can be put on and taken off easily. With some shoe models, you can also adapt the upper width using a hook and loop fastener in the heel area. We recommend the use of crutches to improve gait stability. Additionally, you should practice a safe heel-to-toe movement with the therapeutic shoe.

Due to the indication-based selection, individual adaptation and for hygienic reasons, the therapeutic shoe may only be used by yourself.

Do not cycle or drive a vehicle when wearing a therapeutic shoe.

If there is a considerable difference in height between the sole of the therapeutic shoe and that of the normal shoe, you need a slip-resistant height compensation shoe. In our shoe catalogues, you will find the matching height compensation shoe for your shoe model under the name TWINSHOE.

.....

2.1 Therapeutic Shoes POST-OP

The following therapeutic shoes are part of the FIOR & GENTZ product range for post-op patients:

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MÜNCHEN
OSLO
REGENSBURG
ROSTOCK
STENDAL
STOCKHOLM
STUTTGART
TOKIO
WUPPERTAL

Relief Shoes

The FIOR & GENTZ relief shoes reduce the pressure on the injured area of the foot. Some models can be combined with a protective cap that protects the foot from cold and wet conditions.

The FIOR & GENTZ forefoot relief shoes (example FRANKFURT, fig. 1) provide an open toe area and a wedge sole with a forefoot support or a forefoot rocker. The latter assists the foot's heel-to-toe movement. Forefoot relief shoes without forefoot support can be supplied with a protective stirrup if required.

fig. 1

fig. 2

The FIOR & GENTZ heel relief shoes (example WUPPERTAL, fig. 2) provide an open heel area. Some shoe models also have a shortened sole without heel support which relieves the heel area.

.....

2.2 Therapeutic Shoes DIABETES

The following therapeutic shoes are part of the FIOR & GENTZ product range for diabetic patients:

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fig. 3

The FIOR & GENTZ diabetic shoes (example KASSEL, fig. 3) have removable insoles. This allows you to easily place an individual diabetic foot orthotic into the shoe. Do not select a shoe that is too tight to avoid pressure points and chafing. If more space is required due to bandages, foot deformities, etc., the inner volume can be adapted or a wider shoe version can be selected.

2.3 Therapeutic Shoes HOMECARE

The following therapeutic shoes are part of the FIOR & GENTZ product range for home care patients:

HEIDELBERG
KONSTANZ
LEIPZIG
WEIMAR

fig. 4

The FIOR & GENTZ Homecare shoes (example HEIDELBERG, fig. 4) have a flexible sole and can be worn both inside and outside. They are particularly useful for a stay in a (rehabilitation) clinic or a retirement home.

3. Use

3.1 Putting on the Therapeutic Shoe

Always put on and take off the therapeutic shoe while seated. Unless the foot is protected by a bandage, you should always wear socks in your therapeutic shoe.

-
- 1 Open the therapeutic shoe (fig. 5).
 - 2 Place your foot on the sole (fig. 5).
 - 3 Close the hook and loop fasteners (fig. 6).
- If your therapeutic shoe has a hook and loop fastener in the heel area, close that one at the end (fig. 7–8).

fig. 5

fig. 6

The hook and loop fasteners should fit tightly. However, excessive pressure should be avoided.

fig. 7

fig. 8

3.2 Attaching the Protective Forefoot Cap

- 1 Depending on the model, attach the protective forefoot cap on the inside or the outside of the therapeutic shoe (fig. 9–10).
- 2 Place your foot in the shoe.
- 3 Place the tongue of the protective cap over the foot (fig. 11).
- 4 Then, put the wings of the upper on top of the tongue and close the hook and loop fasteners (fig. 12).

fig. 9

fig. 10

fig. 11

fig. 12

.....

3.3 Attaching the Protective Heel Cap

1. Fix the protective heel cap (fig. 13). With the exception of the WETZLAR model (fig. 14), this should be done before putting on the shoe.
2. Place your foot in the shoe.
3. Close the hook and loop fasteners in the heel area (fig. 15).

fig. 13

fig. 14

fig. 15

4. Useful Life

Note that the useful life depends on the intensity of use as well as your indication which determines the medically adequate period of time the shoe should be used. That is why we recommend to regularly check the therapeutic shoe for external damages.

In the following table you will find our general recommendation for the period of use of all FIOR & GENTZ therapeutic shoes.

Recommended Period of Use	Therapeutic Shoe
6 months	DRESDEN, HEIDELBERG, KASSEL, KONSTANZ, LEIPZIG, NÜRNBERG, POTSDAM, WEIMAR, WETZLAR
4 months	AMSTERDAM, AUGSBURG, BREMEN, ERFURT, FLENSBURG, FRANKFURT, FREIBURG, HAMBURG, HELSINKI, KOPENHAGEN, LÜBECK, MADRID, OSLO, REGENSBURG, ROSTOCK, STUTTGART, TOKIO
3 months*	MAINZ, LONDON

Recommended Period of Use	Therapeutic Shoe
2 months*	BERLIN, BORDEAUX, HANNOVER, KÖLN, LÜNEBURG, MAILAND, MARSEILLE, MÜNCHEN, STENDAL, STOCKHOLM, WUPPERTAL
1 month*	KAISERSLAUTERN

* Please note that the indicated periods for the use of therapeutic shoes with wedge soles only apply if crutches are used.

5. Care

To avoid pressure points, chafing and unnecessary material wear, always check inside your therapeutic shoe for foreign objects like small stones or coarse dirt before wearing it.

5.1 General Information on Cleaning

Clean your therapeutic shoe regularly. Every time you clean the therapeutic shoe, inspect it for signs of wear and have it repaired, if necessary. The therapeutic shoe must not be dried in the dryer or at the radiator (fig. 16). Instead, let it dry slowly at room temperature. After taking off the therapeutic shoe, leave it open so that moisture can escape and no unpleasant odours develop.

fig. 16

.....

5.2 Cleaning

Remove minor soiling with a soft brush or a damp cloth. In case of severe soiling, you can wash the therapeutic shoe in the washing machine at 40°C without spin cycle (fig. 17). Use a laundry net as well as neutral soap or vinegar water. Close the hook and loop fasteners before washing so that they will not become matted. Afterwards, stuff the therapeutic shoe with newspaper and let it dry.

fig. 17

6. Disposal

Therapeutic shoes can be disposed of as residual waste (fig. 18). The therapeutic shoe must be disposed of in particular in the event of damages or severe signs of wear.

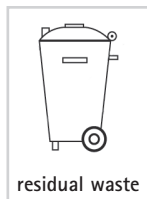
fig. 18

7. CE conformity

We declare that our medical devices as well as our accessories for medical devices are in conformity with the requirements of the Medical Devices Directive 93/42/EEC. Therefore, the FIOR & GENTZ products bear the CE marking.

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