P1

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Content	Page
1. CE Conformity	7
2. Warranty	7
3. Safety Instructions	7
3.1 Classification of the Safety Instructions	7
3.2 Structure of the Safety Instructions	8
3.3 All Instructions for your Safety	8
4. Application	9
5. Functionality	9
6. Putting the P1 Foot Lifter Orthosis on	9
7. Cleaning	9

Dear Patient,

You have received from your orthotist a preproduced **P1 foot lifter orthosis**.

1. CE Conformity

We declare that our medical devices as well as our accessories for medical devices are in conformity with the requirements of Regulation (EU) 2017/745. Therefore, the FIOR & GENTZ products bear the CE marking.

Printed version: 2020-03

2. Warranty

The warranty applies only if the product is used under the described conditions and for the intended purpose. Furthermore, we do not accept a multi-use of the product. Other than that, we refer to our General Terms and Conditions of Business Transactions, Sales, Delivery and Payment.

3. Safety Instructions

In order that your orthosis works in the best way, you can handle it correctly and you can use it as long as possible, please note all information provided in these instructions for use for patients, especially the safety instructions indicated by **ATTENTION!** **CAUTION!** **WARNING!** and **DANGER!** which are listed and explained in the following paragraph. Ignoring this information may lead to injuries and property damage.

3.1 Classification of the Safety Instructions

DANGER!

➔ *Important information about a possible dangerous situation which, if not avoided, leads to death or irreversible injuries.*

WARNING!

➔ *Important information about a possible dangerous situation which, if not avoided, leads to reversible injuries that need medical treatment.*

CAUTION!

➔ *Important information about a possible dangerous situation which, if not avoided, leads to light injuries that do not need medical treatment.*

ATTENTION!

➔ *Important information about a possible situation which, if not avoided, leads to damage of the product.*

All serious incidents according to Regulation (EU) 2017/745 which are related to the product have to be reported to the manufacturer and to the competent authority of the Member State in which the user and/or the patient is established.

3.2 Structure of the Safety Instructions

The following example demonstrates the structure of the safety instructions:

Attention symbol	Signal word	ATTENTION!
	Type of danger	<i>Mechanical load is too high!</i>
	Possible consequences for components	➔ <i>Wear of the joint parts.</i>
	Possible consequences for you	➔ <i>Maintenance intervals shorten.</i>
	Measures	Let your orthotist inform you about maintenance intervals and respect them.

3.3 All Instructions for your Safety

Respect the following safety instructions to avoid injuries and property damage:

WARNING!

Orthosis is put on incorrectly!

➔ *Pressure points and possible loss of fixing function.*

Put the orthosis on as described in these instructions for use.

CAUTION!

Orthosis is not cleaned after wearing it in salt water!

➔ *Corrosion of the riveted joints.*

➔ *Skin irritations and impairment of wound healing.*

Rinse the orthosis with clear water after wearing it in salt water.

CAUTION!

Orthosis is worn directly on the skin!

➔ *Skin irritations and impairment of wound healing.*

Do not wear the orthosis directly on your skin and never wear it on any injured skin.

4. Application

The P1 foot lifter orthosis is exclusively for use for orthotic fittings of the foot.

5. Functionality

The P1 foot lifter orthosis is used for a drop foot. For cases of partial or total failure of the dorsiflexors, the P1 foot lifter orthosis holds the foot in a functional position of 90° (fig. 1).

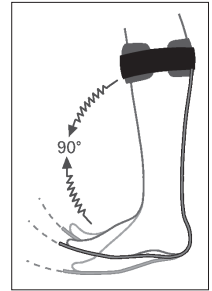
Fig. 1

6. Putting the P1 Foot Lifter Orthosis on

WARNING!

Orthosis is put on incorrectly!

➔ *Pressure points and possible loss of fixing function.*

Put the orthosis on as described in these instructions for use.

1. Unless your foot is protected by a bandage, always wear socks with the P1 foot lifter orthosis.
2. Place the P1 foot lifter orthosis in the shoe. If your shoe has a removable insole, place the P1 foot lifter orthosis underneath (fig. 2). If your shoe has no removable insole, use the provided padding sole.
3. Open the shoe as wide as possible to avoid excessive bending of the spring bar when putting the shoe on (fig. 3) and to avoid any damages to the orthosis.
4. Put the shoe on and tie it.
5. Close the hook and loop fastener of the P1 foot lifter orthosis. The hook and loop fastener has to fit tightly to the leg but must not apply pressure to muscles and soft parts.

Fig. 2

Note:

If you have diabetes, check your feet and leg regularly for pressure points in order to avoid skin injuries. If there are pressure points or abrasions, immediately let your orthotist adapt the orthosis anew.

Fig. 3

7. Cleaning

CAUTION!

Orthosis is not cleaned after wearing it in salt water!

➔ *Corrosion of the riveted joints.*

➔ *Skin irritations and impairment of wound healing.*

Rinse the orthosis with clear water after wearing it in salt water.

The durability of the P1 foot lifter orthosis requires proper handling. Regularly clean the pads and the hook and loop fastener with a gentle detergent. Rinse it with clear water after wearing the P1 foot lifter orthosis in salt water.

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The quality standard of the FIOR & GENTZ company has been controlled and certified by an independent certification organisation according to the international standards ISO 9001 and ISO 13485.

PB1100-PO/ID-2020-03