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Druckversion: 2020-05

Content	Page
<b>1. Information</b>	<b>15</b>
<b>2. Safety Instructions</b>	<b>15</b>
2.1 Classification of the Safety Instructions	15
2.2 All Instructions for a Safe Handling of the System Knee Joint	15
<b>3. Use</b>	<b>17</b>
3.1 Intended Use	17
3.2 Indication	17
3.3 Qualification	17
3.4 Application	17
3.5 Combination Possibilities with Other System Joints	17
<b>4. Joint Function</b>	<b>18</b>
<b>5. Scope of Delivery</b>	<b>18</b>
<b>6. Load Capacity</b>	<b>18</b>
<b>7. Tools for Assembling the System Joint</b>	<b>18</b>
<b>8. Assembly Instructions</b>	<b>19</b>
8.1 Mounting the Extension Stop	19
8.2 Mounting the Joint's Lower Part	19
8.3 Checking the System Joint's Free Movement	20
8.4 Securing the Screws	20
<b>9. Mounting to the System Anchor</b>	<b>20</b>
<b>10. Maintenance</b>	<b>21</b>
10.1 Replacing the Sliding Washers	21
10.2 Dirt Removal	21
<b>11. Spare Parts</b>	<b>22</b>
<b>12. Disposal</b>	<b>24</b>
<b>13. Signs and Symbols</b>	<b>24</b>
<b>14. CE Conformity</b>	<b>24</b>
<b>15. Legal Information</b>	<b>24</b>

---

## 1. Information

These instructions for use are addressed to orthotists or qualified/trained experts and do not contain any notes about dangers which are obvious to them. To achieve maximum safety, please instruct the patient and/or care team in the use and maintenance of the product.

## 2. Safety Instructions

### 2.1 Classification of the Safety Instructions

 <b>DANGER</b>	Important information about a possible dangerous situation which, if not avoided, leads to death or irreversible injuries.
 <b>WARNING</b>	Important information about a possible dangerous situation which, if not avoided, leads to reversible injuries that need medical treatment.
 <b>CAUTION</b>	Important information about a possible dangerous situation which, if not avoided, leads to light injuries that do not need medical treatment.
<i>NOTICE</i>	Important information about a possible situation which, if not avoided, leads to damage of the product.

All serious incidents according to Regulation (EU) 2017/745 which are related to the product have to be reported to the manufacturer and to the competent authority of the Member State in which the orthotist or qualified/trained expert and/or the patient is established.

### 2.2 All Instructions for a Safe Handling of the System Knee Joint

#### **DANGER**

##### **Potential Traffic Accident Due to Limited Driving Ability**

Advise the patient to gather information about all safety and security issues before driving a motor vehicle with orthosis. The patient should be able to drive a motor vehicle safely.

#### **WARNING**

##### **Risk of Falling Due to Improper Handling**

Inform the patient about the correct use of the system joint and possible dangers, especially with regards to excessive mechanical stress (e.g. due to sports, increased activity or weight gain).

---

## WARNING

### **Risk of Falling Due to Loosening of the Bearing Nuts**

Secure the screw of the bearing nut with the specified torque and the corresponding adhesive and make sure that no sliding washers are damaged in the process.

## WARNING

### **Risk of Falling Due to Permanent Higher Load**

If patient data has changed (e.g. due to weight gain, growth or increased activity), recalculate the load capacity of the system joint. For this purpose, use the Orthosis Configurator or contact Technical Support.

## WARNING

### **Risk of Falling Due to Improper Shoe/Wrong Shoe Pitch**

Advise the patient to wear a shoe to which the orthosis is adjusted in order to avoid joint dysfunction.

## WARNING

### **Damage to the Anatomical Joint Due to Incorrect Position of the Joint's Mechanical Pivot Point**

Determine the joint's mechanical pivot points correctly in order to avoid a permanent incorrect load on the anatomical joint. Please refer to the online tutorials on our website or contact Technical Support.

## WARNING

### **Jeopardising the Therapy Goal by Not Providing the Necessary Free Movement**

Check if the system joint moves freely in order to avoid restrictions of the joint function. Use suitable sliding washers according to the information in these instructions for use.

## NOTICE

### **Limitation of the Joint Function Due to Improper Processing**

Errors in processing can impair the joint function. Pay particular attention to:

- correctly connect the system anchor with the system joint in accordance with the production technique;
- grease the joint components only slightly and
- adhere to the maintenance intervals.

---

### 3. Use

#### 3.1 Intended Use

The **NEURO CLASSIC H<sub>2</sub>O** system knee joint is exclusively for use for orthotic fittings of the lower extremity. The system joint provides motion control and is only allowed to be used for producing a KAFO. Every system joint influences the orthosis' function and thus also the function of the leg. Depending on the type of orthotic fitting and the patient's individual physical abilities, driving a motor vehicle safely should be guaranteed.

#### 3.2 Indication

The indication for the treatment with an orthosis for the lower extremity is a pathological gait. This can be caused, for example, by central, peripheral, spinal or neuromuscular paralyses, structurally conditioned deformities/malfunctions or surgery.

The physical conditions of the patient such as muscle strength or activity level are decisive for the orthotic treatment. A safe handling of the orthosis must be ensured.

#### 3.3 Qualification

The system joint must be handled by an orthotist or qualified/trained expert.

#### 3.4 Application

All **FIOR & GENTZ** system joints were developed for everyday life activities such as standing and walking. Extreme loads connected to activities like running, climbing and parachuting are excluded. The **NEURO CLASSIC H<sub>2</sub>O** system knee joint is waterproof and, therefore, it is suited for a usage in wet areas.

#### 3.5 Combination Possibilities with Other System Joints

The **NEURO CLASSIC H<sub>2</sub>O** system knee joint can be mounted in combination with waterproof system ankle joints from our product range in a waterproof orthosis. It can also be used as supporting joint for the **NEURO LOCK H<sub>2</sub>O** system knee joint.

For selecting all system components for your orthosis, be sure to use the Orthosis Configurator and mind the configuration result.

## 4. Joint Function

The NEURO CLASSIC H<sub>2</sub>O is a free moving, monocentric system knee joint to provide motion control. It is preassembled in a physiological joint angle of 5°.

The system joint also has an integrated posterior offset (fig. 1).

System Width	20mm
Posterior Offset of the Joint Axis	20mm

## 5. Scope of Delivery

Description	Quantity
system knee joint (without figure)	1
set 2-component adhesive with primer (fig. 2)	1
assembly/lamination dummy (fig. 3)	1

fig. 1

fig. 2

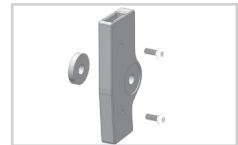
fig. 3

## 6. Load Capacity

The load capacity results from the relevant patient data and can be determined by using the Orthosis Configurator. Use the system components determined by the Orthosis Configurator when producing an orthosis and mind the recommended production technique.

## 7. Tools for Assembling the System Joint

Tools	System Width
	20mm
T8 hexalobular screwdriver/bit	x
T20 hexalobular screwdriver/bit	x
torque screwdriver 1–6Nm	x
sliding washer centring pin	x

## 8. Assembly Instructions

The system joint is delivered fully assembled. All functions are checked beforehand. You have to disassemble the system joint for mounting it in the orthosis and for maintenance. To ensure an optimal functioning, follow the assembly instructions below. Secure the screw with the torque specified in paragraph 8.4.

When mounting the system joint, mind the correct alignment of the entire orthosis as it is essential for the later function of the orthosis.

### 8.1 Mounting the Extension Stop

- 1 Stick the cheese head screw (1; fig. 4) into the extension stop (2).
- 2 Place the pressure spring (3) onto the cheese head screw (1).
- 3 Screw the sub-assembly into the joint's lower part.

fig. 4

### 8.2 Mounting the Joint's Lower Part

- 1 Grease the two sliding washers **slightly** on both sides with orthosis joint grease.
- 2 Place the sliding washers on both sides of the joint's lower part. Do not place the sliding washers centrally, but slightly above the bore (fig. 5).
- 3 Slide the joint's lower part from below into the joint's upper part (fig. 6). Make sure that the sliding washers are centred correctly (fig. 7). To do so, use the sliding washer centring pin.
- 4 Grease the axle bore of the joint axis and the friction surfaces of the bearing nut with orthosis joint grease.
- 5 Put the bearing nut into the opening of the joint's upper part. Make sure that the cut-outs of the bearing nut and the joint's upper part match (fig. 8). The bearing nut must be fully inserted in the opening.
- 6 Place the cover disc onto the front of the joint's upper part. The cover disc must be fully inserted in the opening. Make sure that the cut-outs of the cover disc and the joint's upper part match.
- 7 Screw in the countersunk flat head screw (S1; fig. 9).

fig. 5

fig. 6

fig. 7

fig. 8

fig. 9

### 8.3 Checking the System Joint's Free Movement

Check if the system joint moves freely. If the system joint runs with lateral play, mount the next thicker sliding washer. If it does not move freely (it is jammed), mount the next thinner sliding washer.

### 8.4 Securing the Screws

The screw is secured after the orthosis has been produced and tried on and before it is handed over to the patient.

- 1 Secure the screw for the joint's upper part (fig. 9) with the torque corresponding to the system width and LOCTITE® 243 medium strength.
- 2 Let the adhesive harden (final strength after approx. 24 hours).

Screw for Joint's Upper Part	System Width
S1 (screw 1)	20mm
	4Nm

The screw of the joint's upper part is not secured with the necessary torque at delivery. You can also find information on the torque on the cover disc of the system joint.

## 9. Mounting to the System Anchor

The system anchor of the NEURO CLASSIC H<sub>2</sub>O system knee joint has to be adhered to the system joint according to our recommended production technique (fig. 10). You will find more detailed information in the *Instructions for Use for Orthotists or Qualified/Trained Experts System Anchors for Waterproof System Joints*. You will find information on the production techniques in the section "Online Tutorials" on our website [www.fior-gentz.com](http://www.fior-gentz.com).

fig. 10

---

## 10. Maintenance

Check the system joint for wear and functionality every **6 months**. Also check the functionality after every maintenance carried out.

Joint Component	Problem	Measure
extension stop face	wear	replacing extension stop
sliding bushing	wear	replacing sliding bushing
sliding washer	wear	replacing sliding washer, see paragraph 10.1
bearing nut	wear	replacing bearing nut

Secure the screw for the joint's upper part with the torque corresponding to the system width and LOCTITE® 243 medium strength during every maintenance (see paragraph 8.4). Remove all adhesive residues first.

### 10.1 Replacing the Sliding Washers

Sliding washers are available in different thicknesses (e.g. GS2413-040 is 0.40mm thick). Each thickness has a different marking (fig. 11). You will find the article numbers of the premounted sliding washers on the back page of these instructions for use.

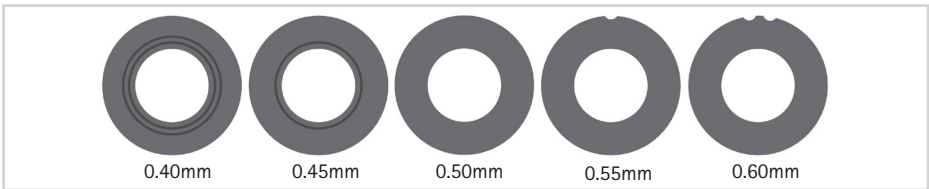
fig. 11

### 10.2 Dirt Removal

The **NEURO CLASSIC H<sub>2</sub>O** system knee joint is suited for a usage in wet areas. Nevertheless, dirt must be removed when necessary and during regular maintenance. For this purpose, disassemble the system joint and clean the soiled system components with a dry cloth.

In order to optimise the lifespan, we recommend rinsing the orthosis with water, especially after using it in salt water, chlorine water and sand.

11. Spare Parts

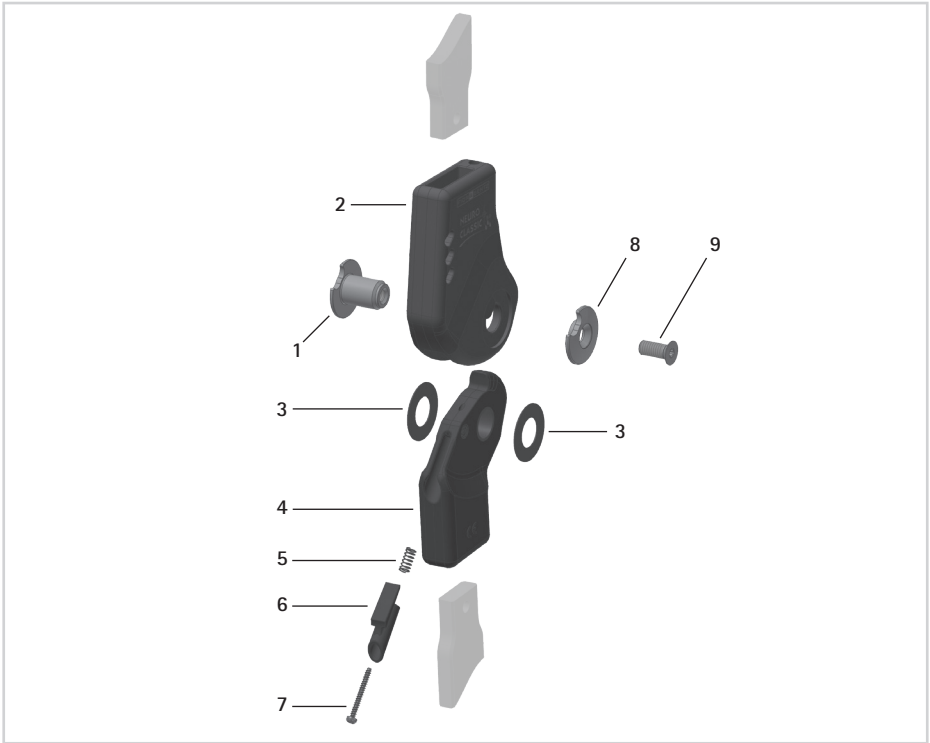
fig. 12

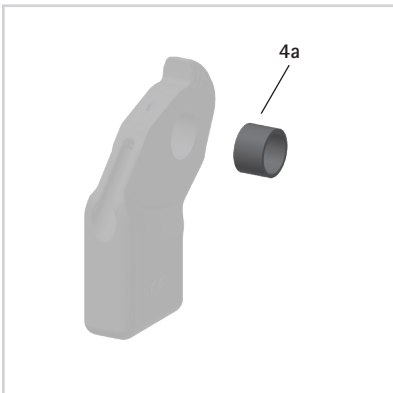
fig. 13

Item	Article Number for System Width	
	20mm	Description
1	SK0895-1/C	bearing nut
2	SL0105-C	upper part, straight, carbon
3	GS2413-*	sliding washer*
4	SK0815-2/C	5° lower part with sliding bushing, straight, carbon-titanium hybrid
4a	BP1412-L090	sliding bushing
5	FE1510-02	pressure spring
6	SK9805-E005/C	5° extension stop
7	SC4402-L22	cheese head screw
8	SK0895-2/C	cover disc
9	SC1406-L14	countersunk flat head screw with hexalobular socket

* Sliding Washers	
Article Number for System Width	
20mm	
Ø = 24mm	
GS2413-040	
GS2413-045	
GS2413-050	
GS2413-055	
GS2413-060	

## 12. Disposal

Dispose of the system joint and its individual parts properly. The product must not be disposed of with the residual waste (fig. 14). Please comply with the applicable national laws and local regulations for the proper recycling of recyclable materials.

fig. 14

For proper disposal, it is necessary to demount the system joint from the orthosis.

## 13. Signs and Symbols

Symbols on the Packaging

medical device

## 14. CE Conformity

We declare that our medical devices as well as our accessories for medical devices are in conformity with the requirements of Regulation (EU) 2017/745. Therefore, the FIOR & GENTZ products bear the CE marking.

## 15. Legal Information

With the purchase of this product, our General Terms and Conditions of Business Transactions, Sales, Delivery and Payment will apply. The warranty expires, for example, if the product is mounted several times. Please note that the product is not supposed to be combined with other components or materials than with those recommended by the FIOR & GENTZ Orthosis Configurator. Should the product be combined with products from other manufacturers, a written consent by the seller is required.

The information in these instructions for use is valid at the date of printing. The contained product information serves as a guideline. Subject to technical modifications.

All rights, particularly the distribution, copy and translation of this manual or any part of it, in paper or as electronic document, must be authorised in writing by FIOR & GENTZ Gesellschaft für Entwicklung und Vertrieb von orthopädiotechnischen Systemen mbH. Reprints, copies and any other electronic reproduction, even partial, must be authorised in writing by FIOR & GENTZ Gesellschaft für Entwicklung und Vertrieb von orthopädiotechnischen Systemen mbH.

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